

# emdin | instructions

## GYPSUM HARDENER

Special Liquid for use with  
**SUPERSTONE DIE STONE AND LAB MODEL STONE**  
**FOR INCREASED HARDNESS AND SURFACE SMOOTHNESS**

*DO NOT FREEZE LIQUID, DISCARD IF FROZEN.*

Use EMDIN's Gypsum hardener liquid instead of water when mixing the die or model stone. This will increase the surface hardness and smoothness of the cast. Use of this liquid also improve the mixability of the stones and the fluidity of the mixed material so that the mix would flow into the impression more smoothly and thereby reducing the entrapment of air bubbles.

### **STORAGE AND HANDLING**

Store in a cool and dry place. Do not allow the liquid to freeze. If the bottle shows sediments on the bottom it indicates the liquid had frozen and it should not be used.

Manufactured by:

**EMDIN INTERNATIONAL CORPORATION**  
15841 Business Center Drive, Irwindale, USA  
TEL: 626-813-3740 FAX: 626-445-5233  
Email: [info@emdin.com](mailto:info@emdin.com)  
Website: <http://www.emdin.com>