

# DEBUBBLIZER BLUE

WATER-BASED SURFACE TENSION REDUCING AGENT

## APPLICATION

Debubbler Blue is a surfactant solution formulated to gently reduce the surface tension of impressions, prior to pouring models, and of wax patterns, prior to investing. Treating impressions and wax patterns with Debubbler Blue also aids in the removal of dust and oily residues accumulated over time. Together, these features allow for better flow and adaptation of the dental material being poured, virtually eliminating the formation of bubbles and voids on the surfaces of models and castings. Debubbler Blue is intended for use when mixing by hand and is available in refillable fine-mist pump spray bottles. Does not contain alcohol.

## DIRECTIONS

1. Remove gross debris from impression / wax pattern.
2. Apply sparingly with gentle brush or fine-mist pump spray bottle.
3. Remove excess surfactant by gently shaking.
4. Pour gypsum / investment under gentle vibration.

## CAUTION!

Keep closed after use. Read instructions before use.

EMDIN INTERNATIONAL CORPORATION  
15841 Business Center Drive, Irwindale, CA 91706  
Tel: (626) 813-3740 Fax: (626) 445-5233  
E-mail: [info@emdin.com](mailto:info@emdin.com) Website: [www.emdin.com](http://www.emdin.com)

**EMDIN**<sup>™</sup>  
INTERNATIONAL CORPORATION

© Emdin International Corporation, Rev. 9/2010