

CERAFILM

DIE LUBRICANT

APPLICATION

Cerafilm is a separating solution formulated for use between die materials and wax patterns. This die lubricant forms a thin film that adapts uniformly to the surface of dies without causing detail degradation. Its non-oily, water-soluble, design allows for a residue-free removal while maintaining dimensional stability. Cerafilm is suited for use with stone, epoxy, metal, and will not react with wax patterns or investment materials.

DIRECTIONS

1. Apply sparingly to die with brush prior to wax up procedures.
2. Remove excess die lubricant with a gently stream of air.
3. Allow die to rest briefly before buildup of the wax pattern.
4. Wax pattern will now easily and cleanly separate from the die.

CAUTION!

Use mask and avoid inhalation. Keep closed after use. Do not freeze. Do not use after thawing if frozen. Shake well. Read instructions before use.

EMDIN INTERNATIONAL CORPORATION
15841 Business Center Drive, Irwindale, CA 91706
Tel: (626) 813-3740 Fax: (626) 445-5233
E-mail: info@emdin.com Website: www.emdin.com

EMDIN[™]
INTERNATIONAL CORPORATION

© Emdin International Corporation, Rev. 9/2010